


# ICAROS GUARDIAN ASSEMBLY INSTRUCTIONS

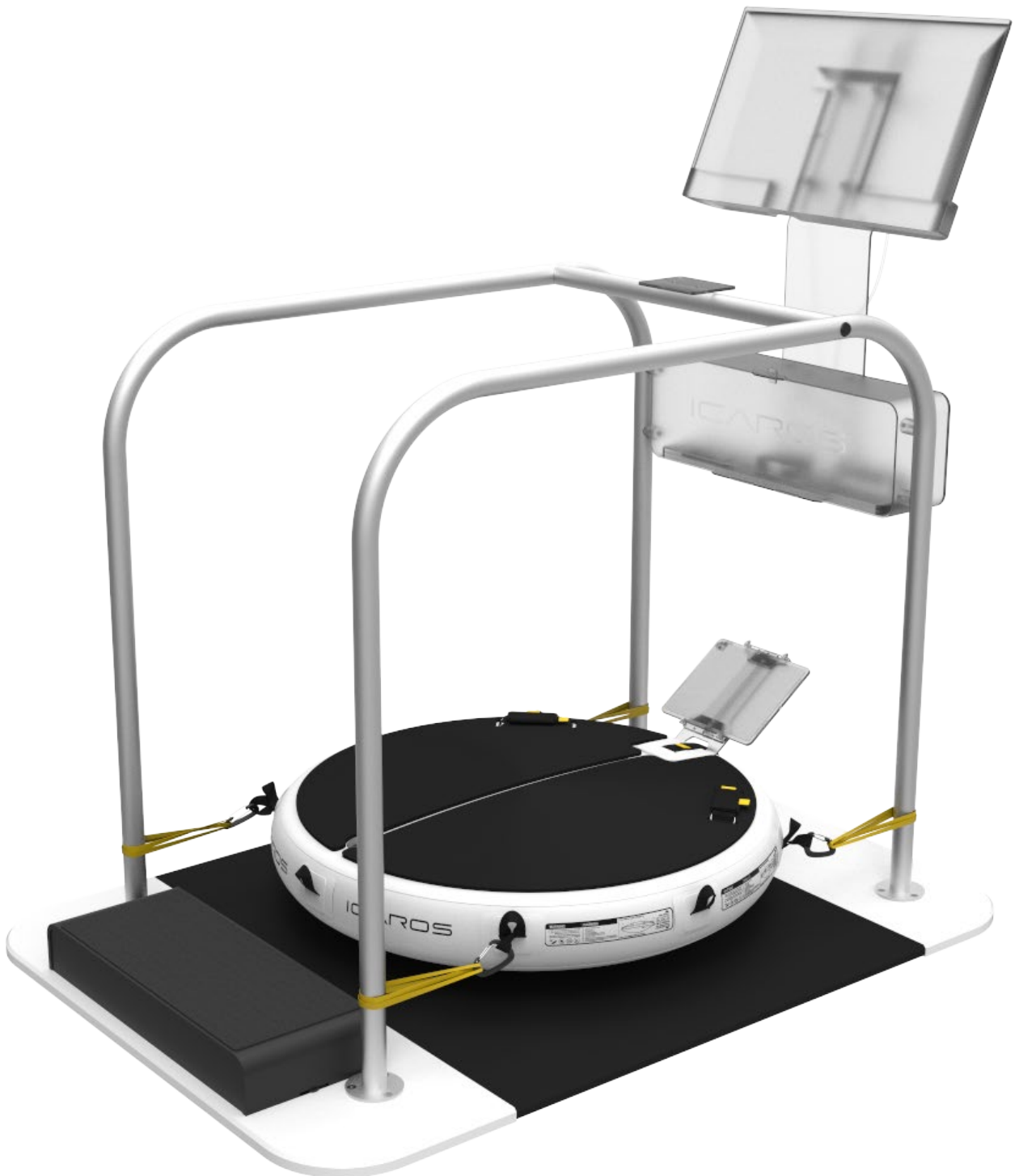

# Contact Details

## Manufacturer

**ICAROS GmbH**  
Fraunhoferstraße 5  
82152 Martinsried  
GERMANY



TEL +49 89 414 1821 00  
MAIL [info@icaros.com](mailto:info@icaros.com)  
WEB [www.icaros.com](http://www.icaros.com)

## Distributor

**ICAROS GmbH**  
Fraunhoferstraße 5  
82152 Martinsried  
GERMANY

TEL +49 89 414 1821 00  
MAIL [info@icaros.com](mailto:info@icaros.com)  
WEB [www.icaros.com](http://www.icaros.com)




# Scope of delivery


## ICAROS Guardian




## Digital Accessoires

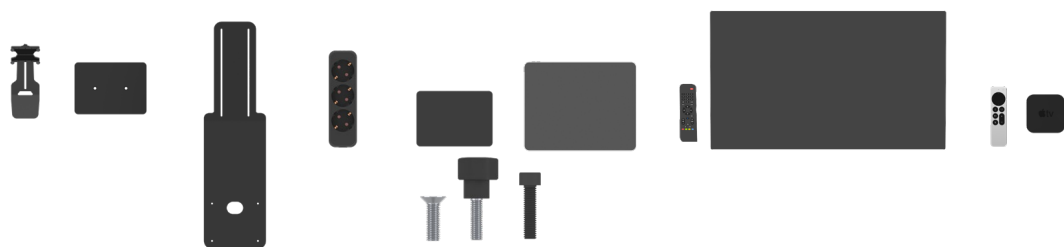

## Assembly Requirements

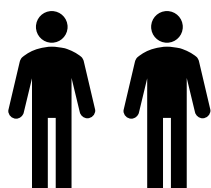

2 Persons

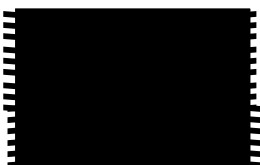


Soft Underlay




Scissors

# ICAROS Guardian

This part of the manual guides you through the setup of the ICAROS Guardian.

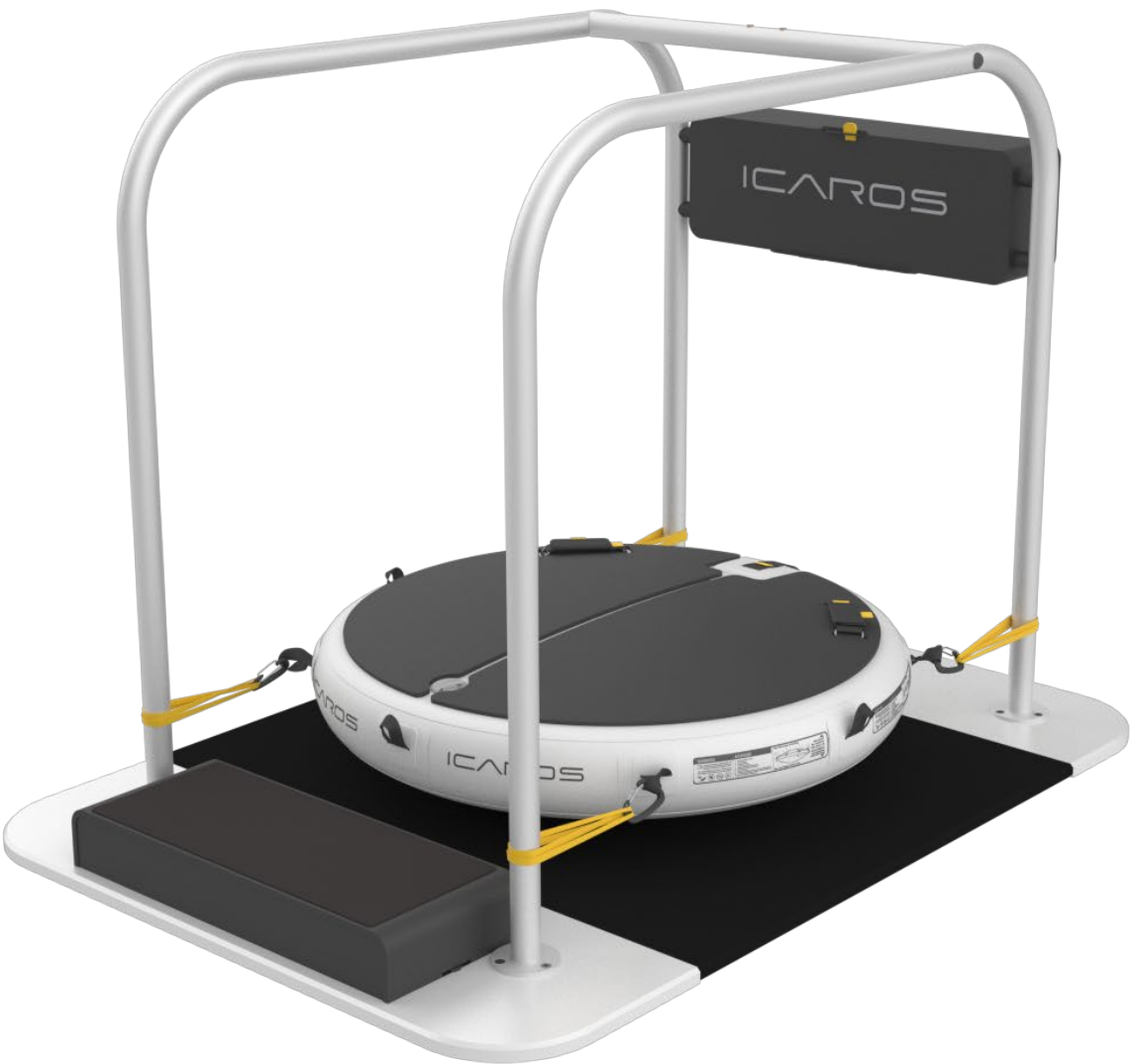


1.1

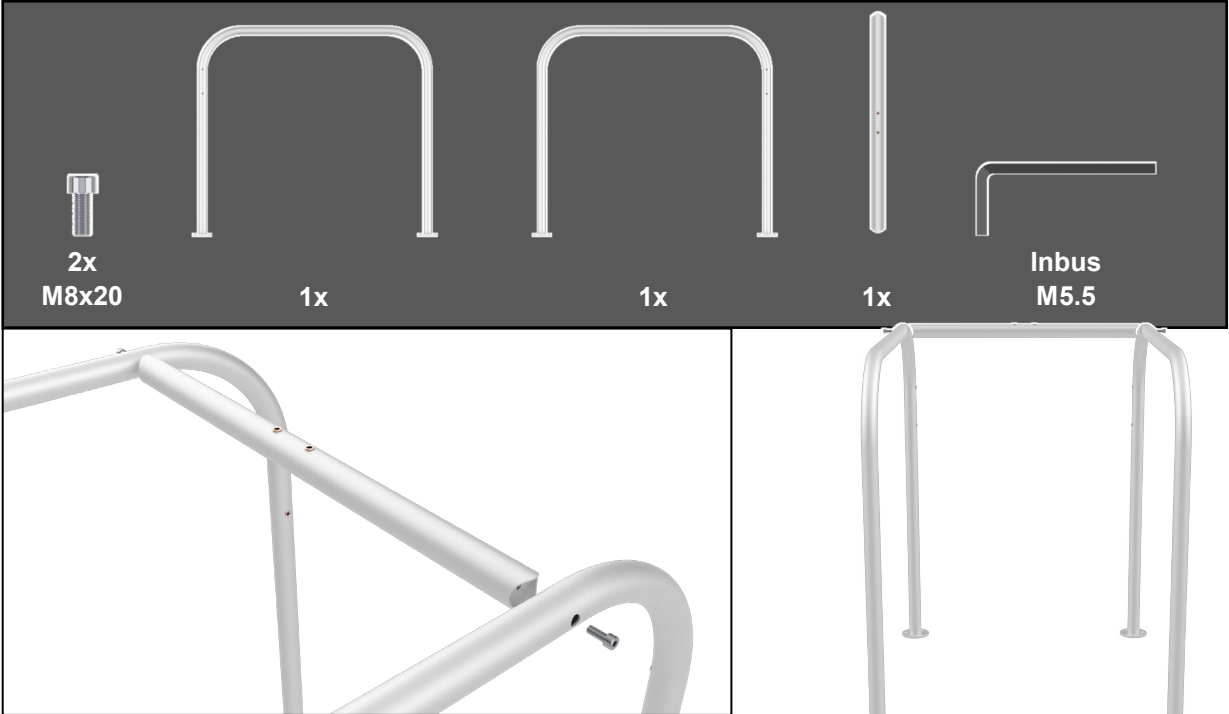

Take care not to tighten the screws so that you can insert the box more easily in the next step.


1.2

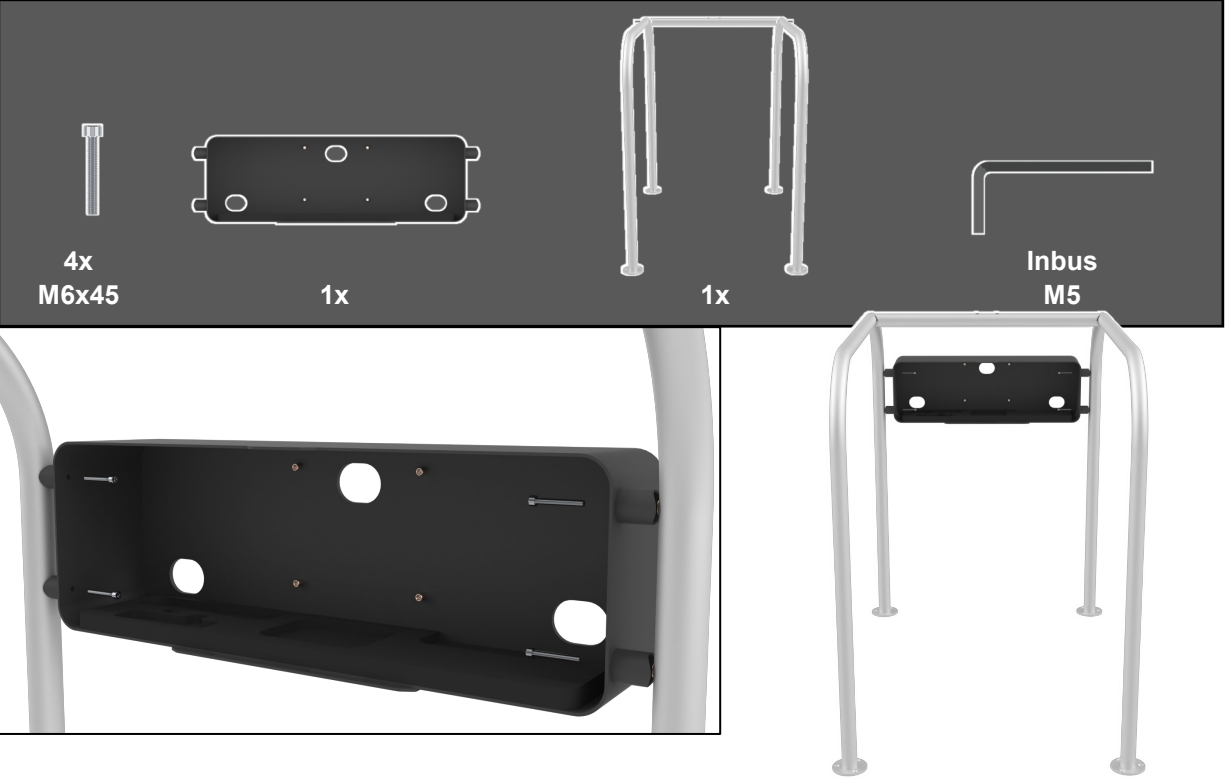


1.3




1.4

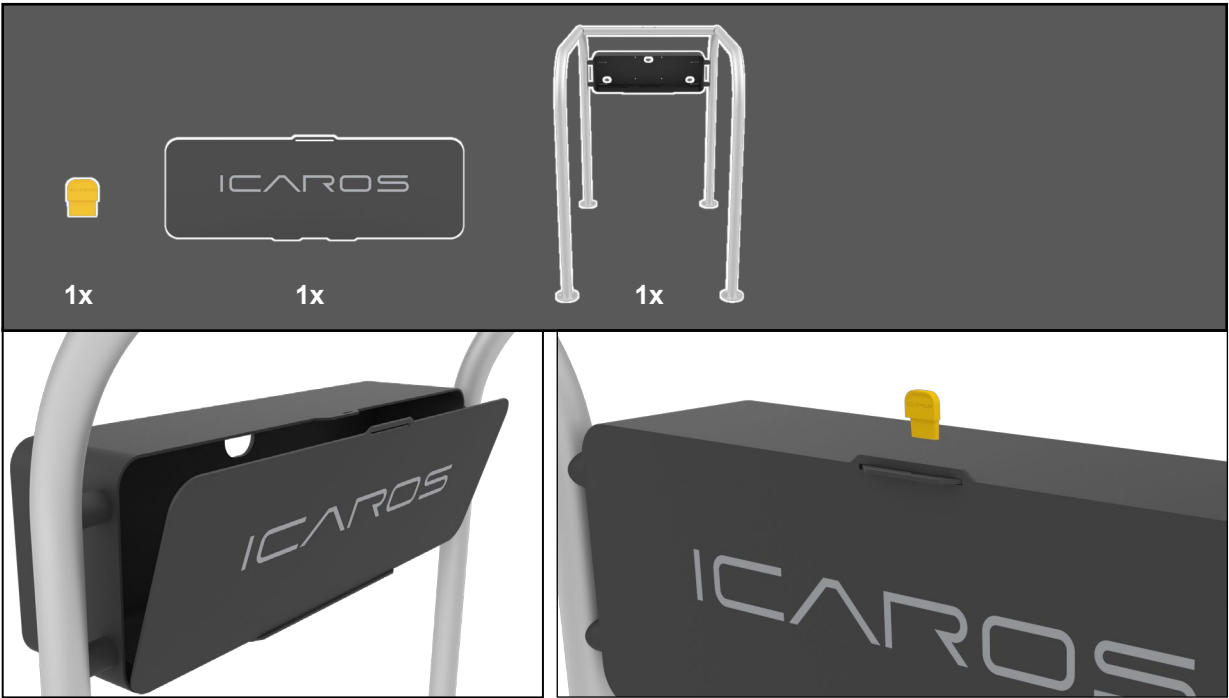


1.5

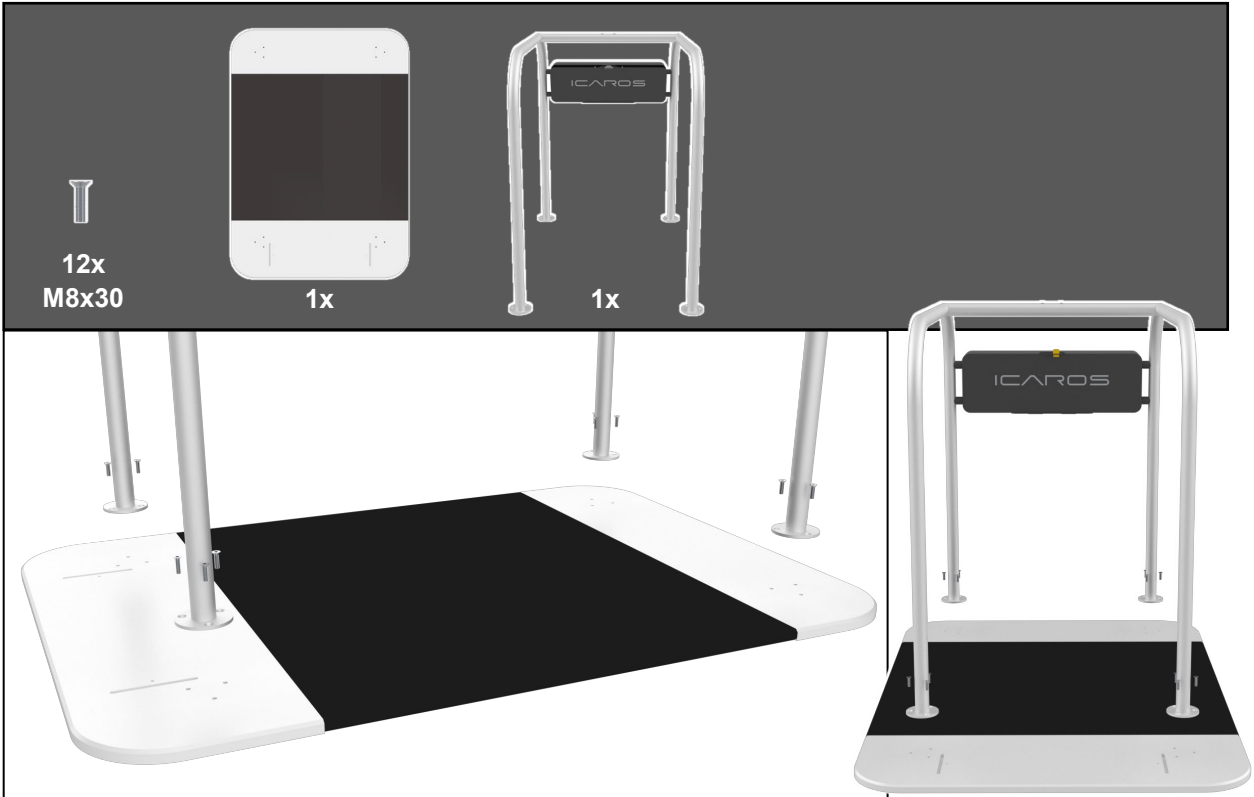


1.6

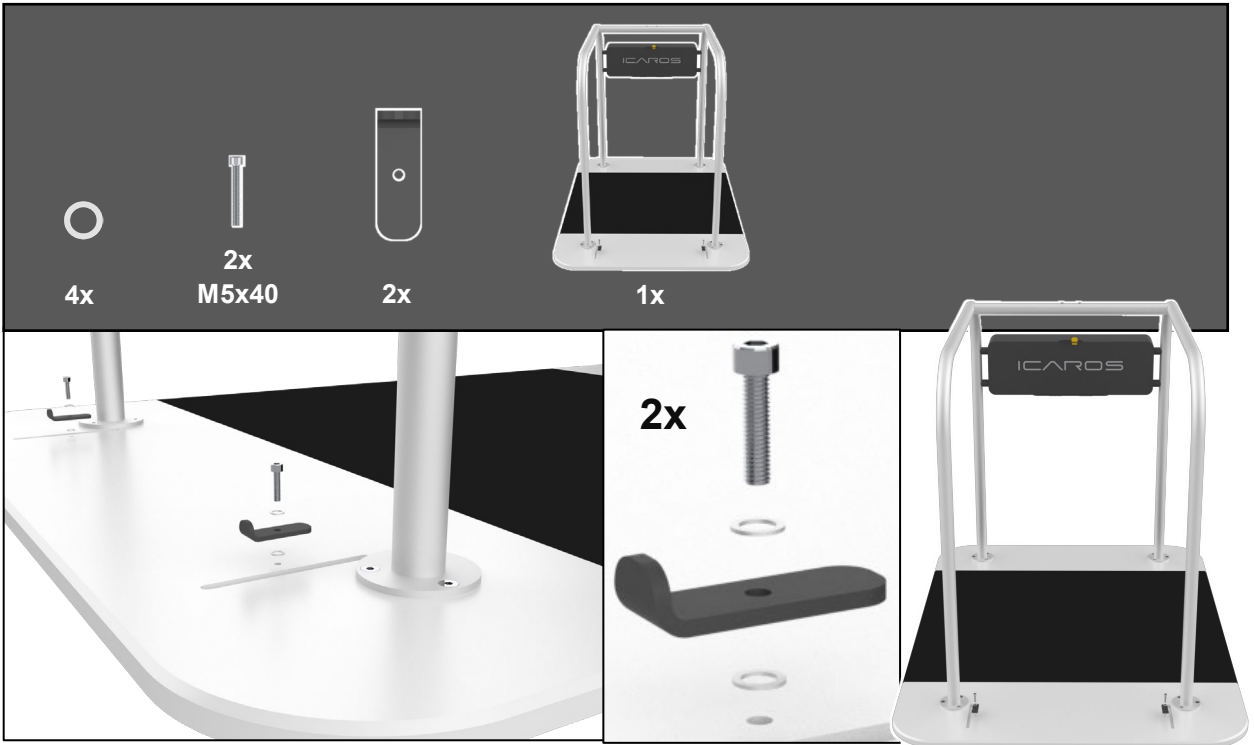

## 1.7

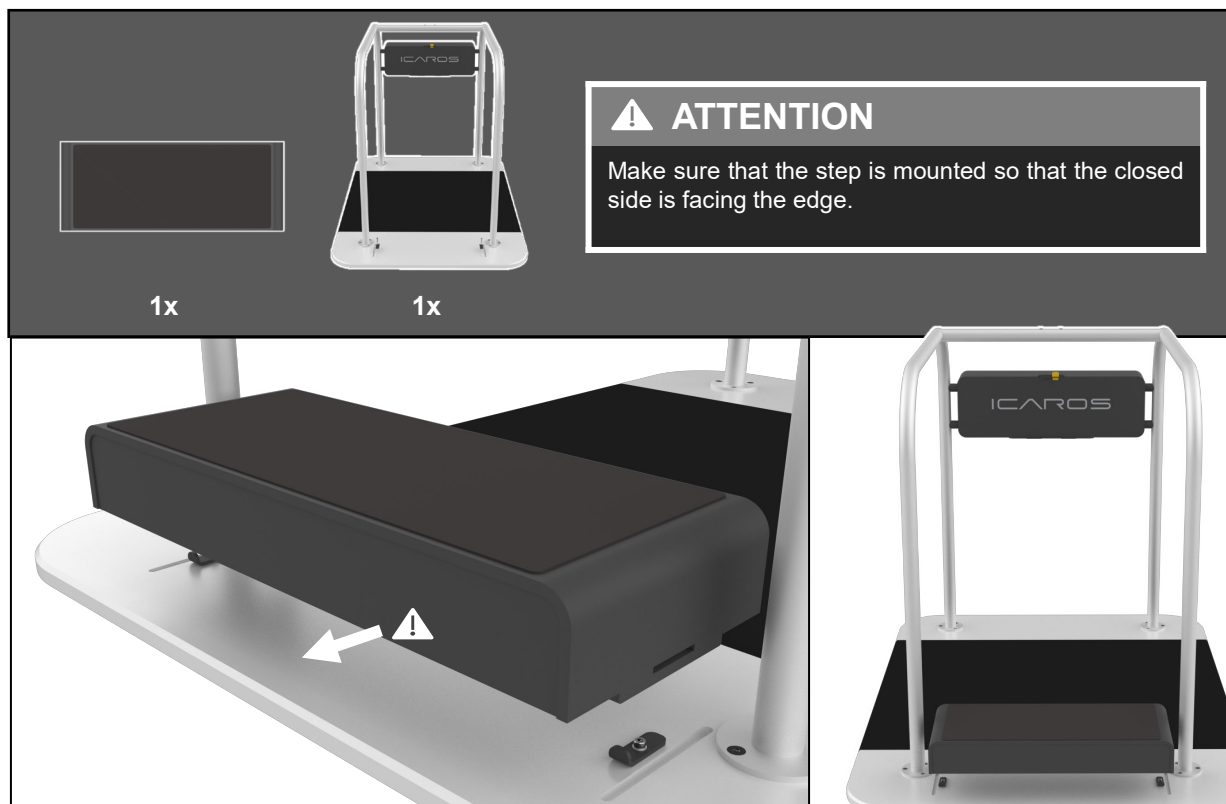

## 1.8

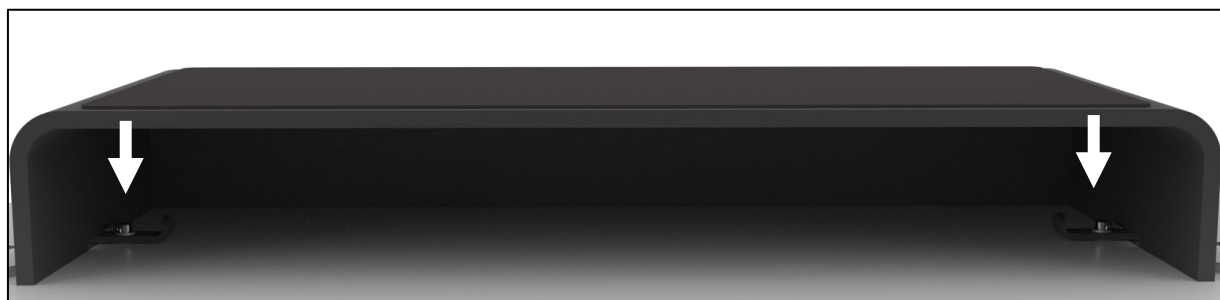


Rotate the levers inside the step for 90° in order to lock it.



## 1.9




1x

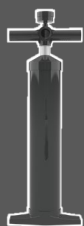

1x


### ⚠ ATTENTION

First inflate the upper and then the lower air chamber. Never exceed the specified air pressure values for the respective chamber.

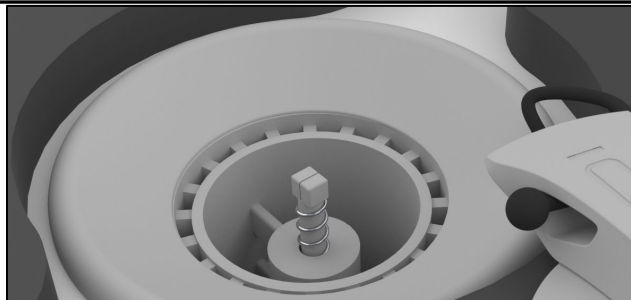

### ⚠ ATTENTION

The valve must be closed before inflation. When the push button is extended, the valve is closed. To open or close the valve, press and turn the push button.

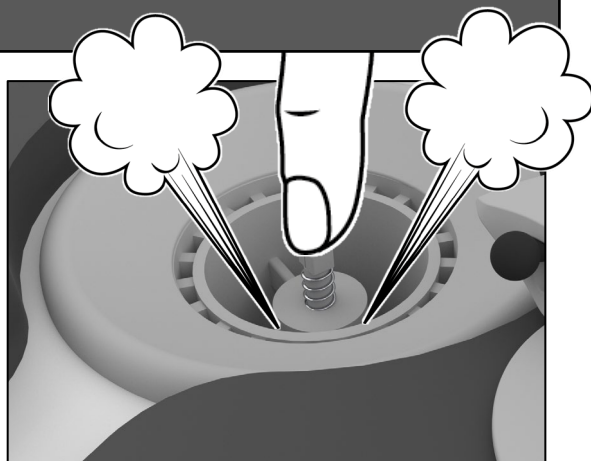

Press the push button down to allow the air to escape.

### 1.9.1



Inflate upper air chamber:

Min.: 0,7 bar

Max.: 1 bar

### 1.9.2






Inflate lower air chamber:

Min.: 0,01 bar

Max.: 0,05 bar

1.10

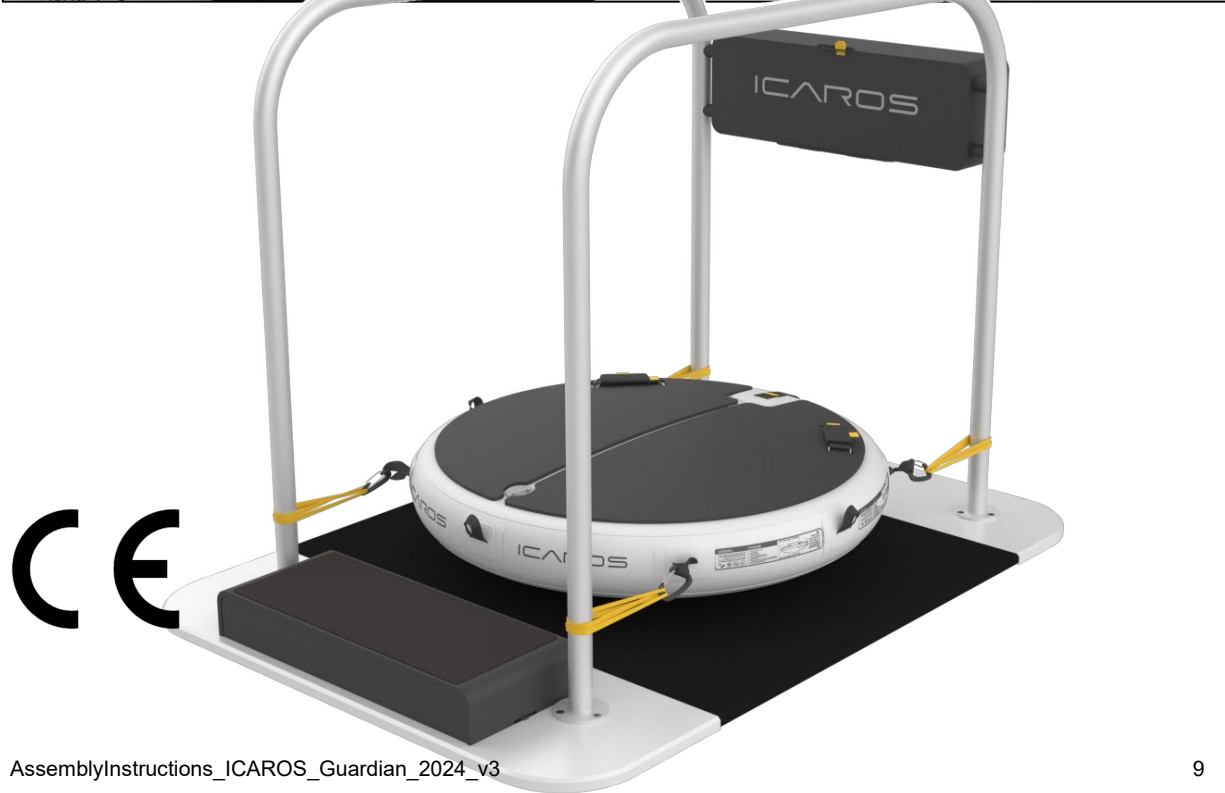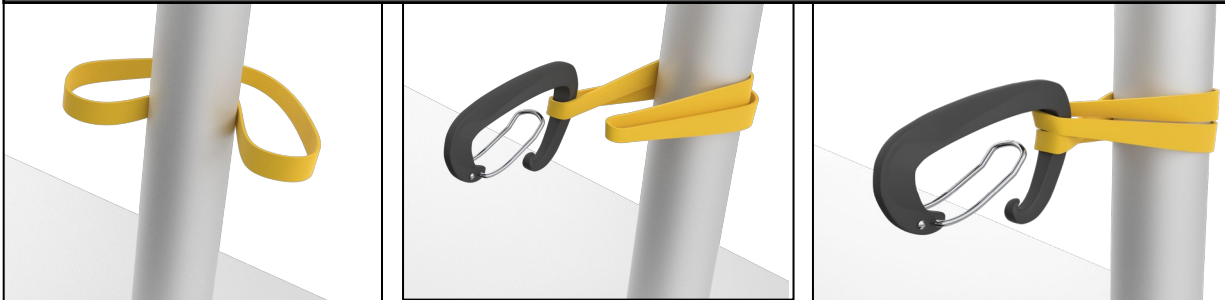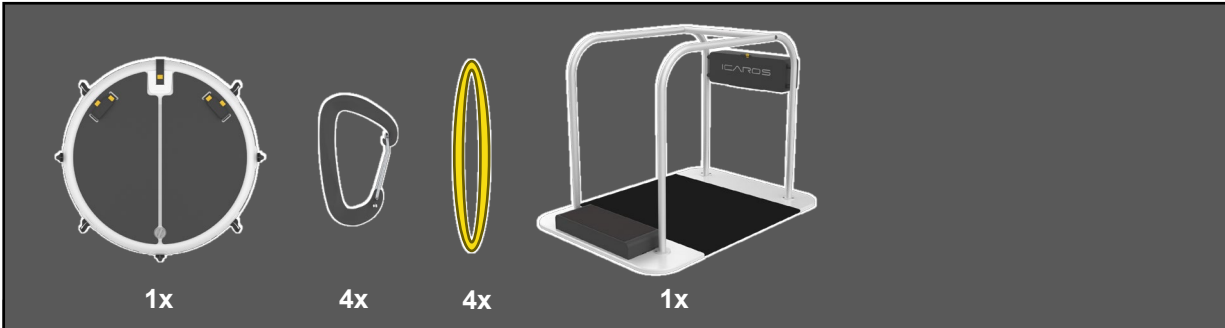


## 2 ICAROS Guardian Digital Accessoires


This part of the manual will guide you through the setup of the digitally enhanced ICAROS Guardian. Please note that the CE certification only covers the ICAROS Guardian without digital extension.




# 2.1

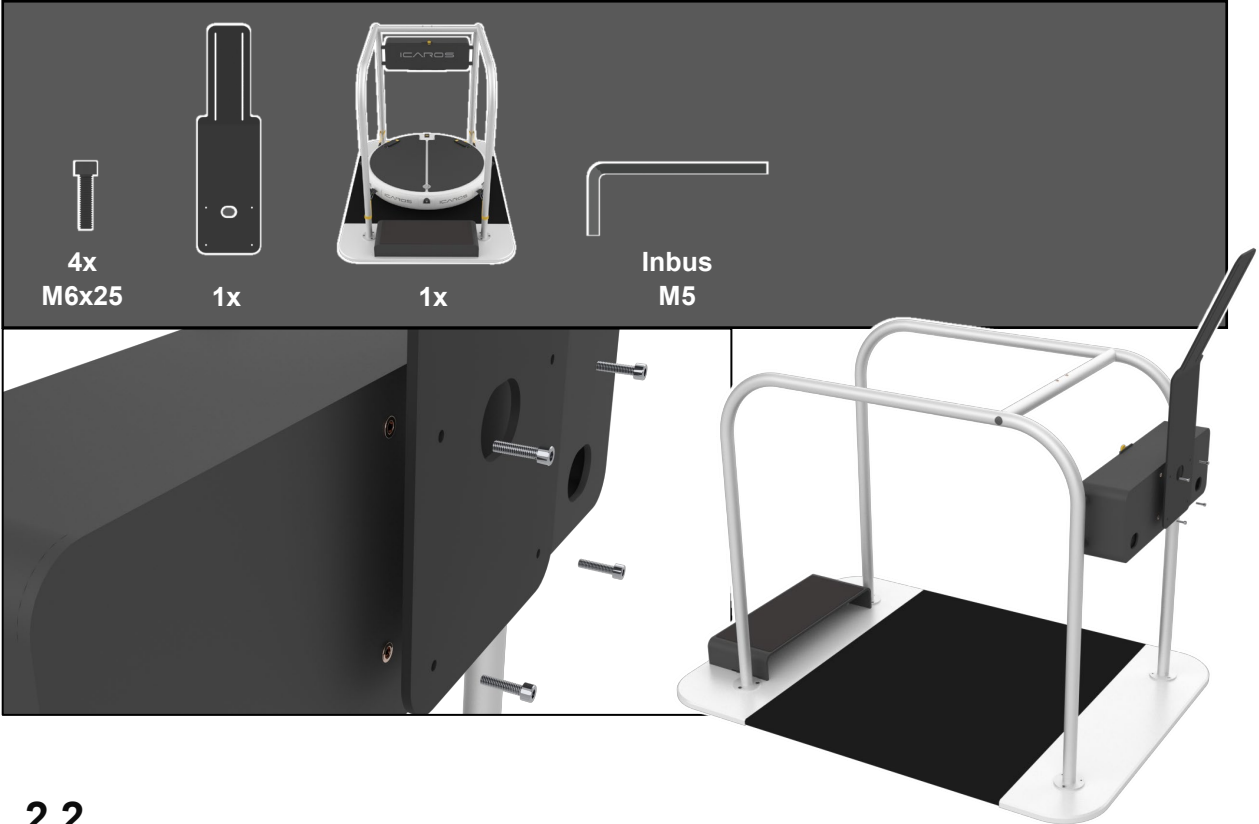


# 2.2

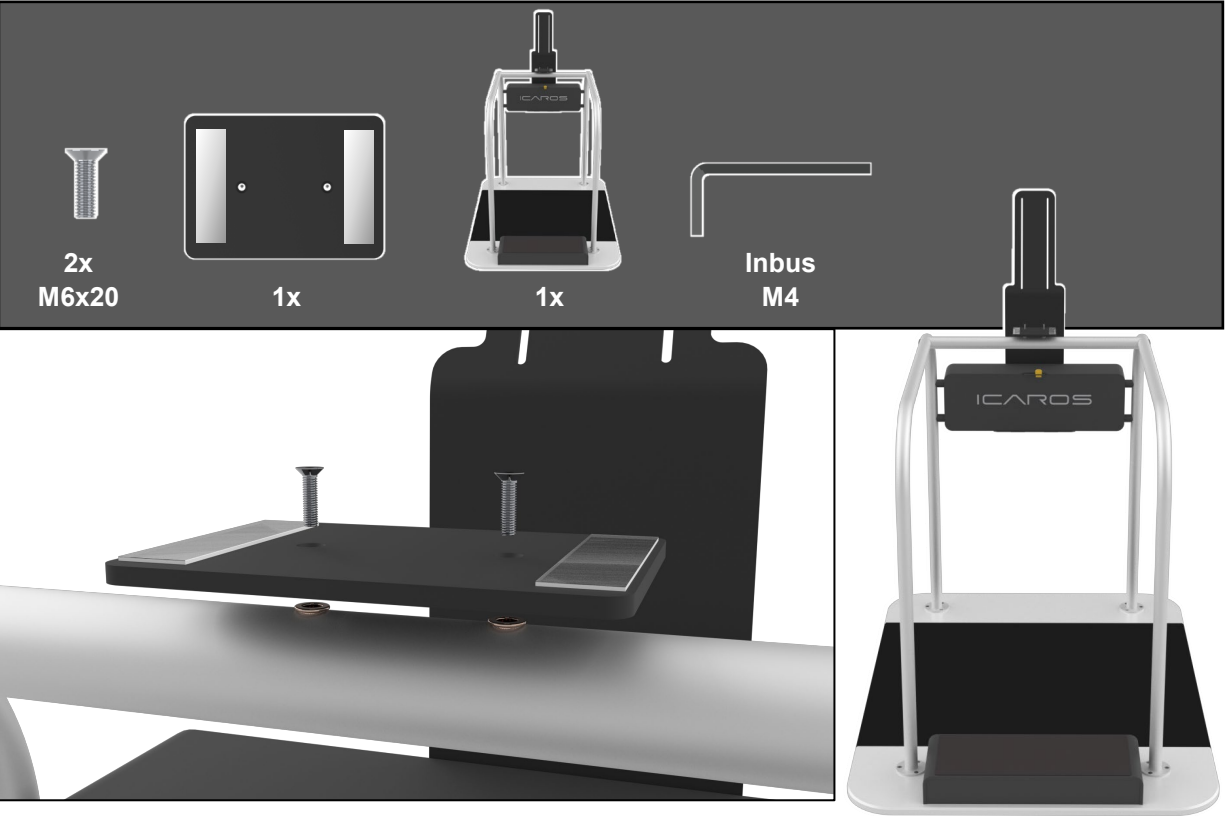





2.3

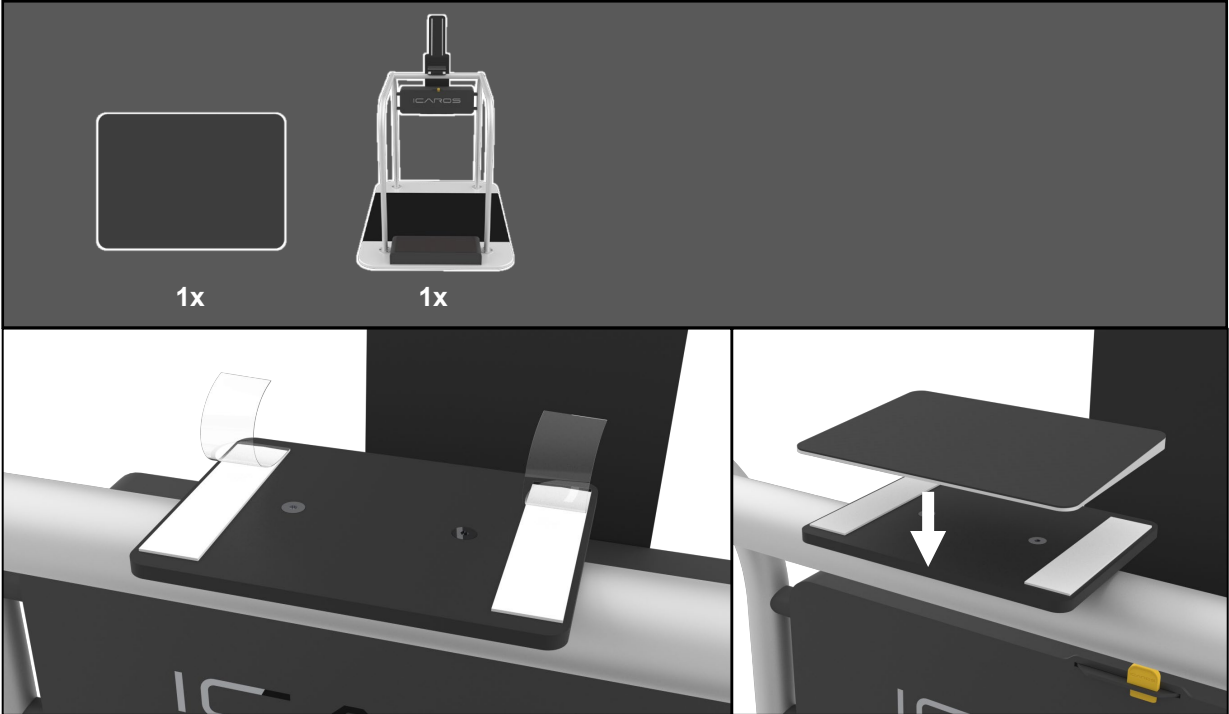



2.4

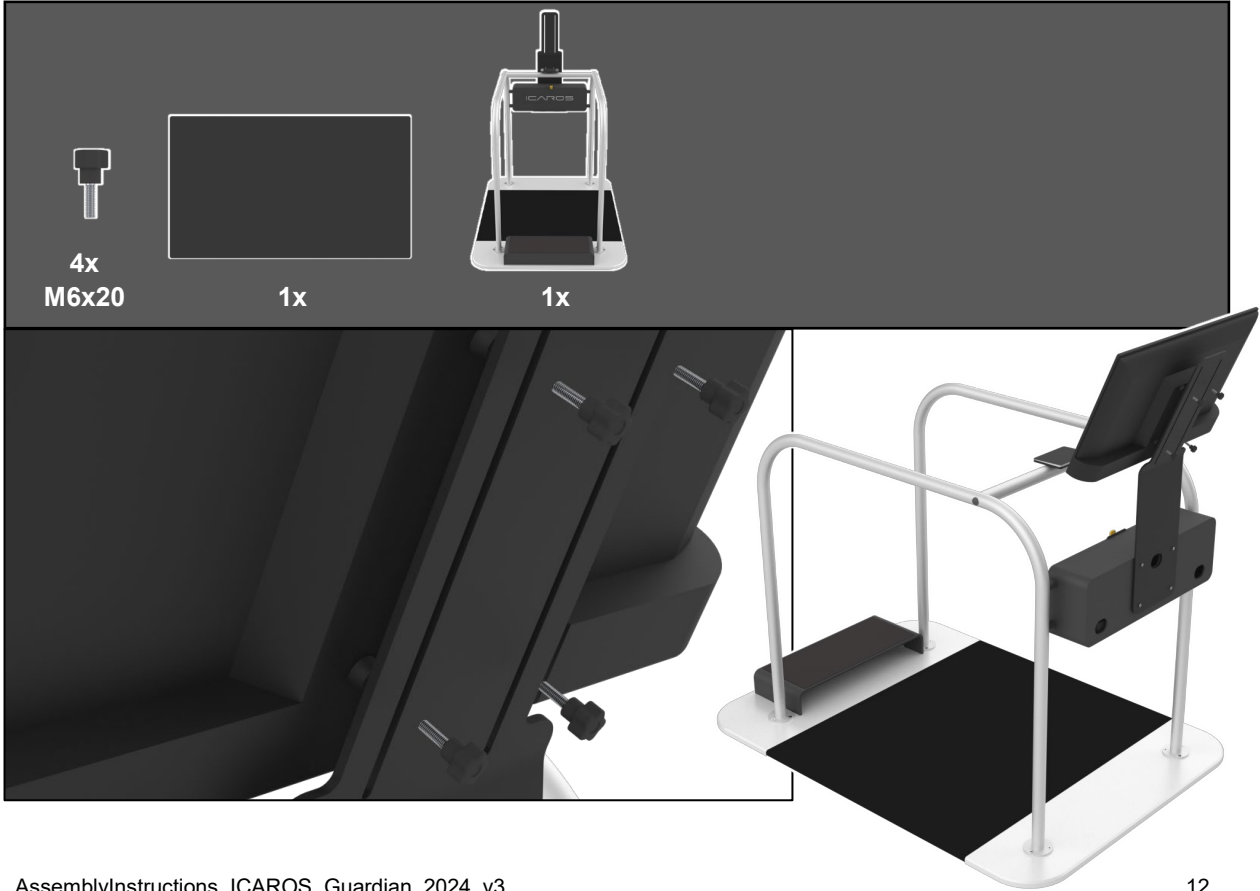

2.5

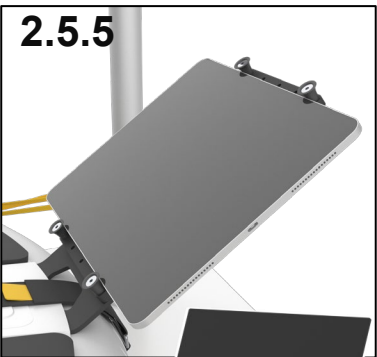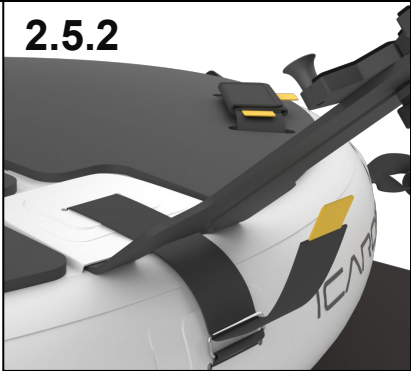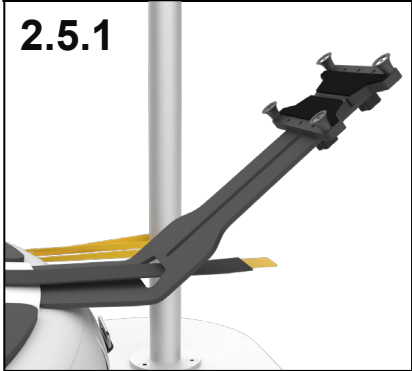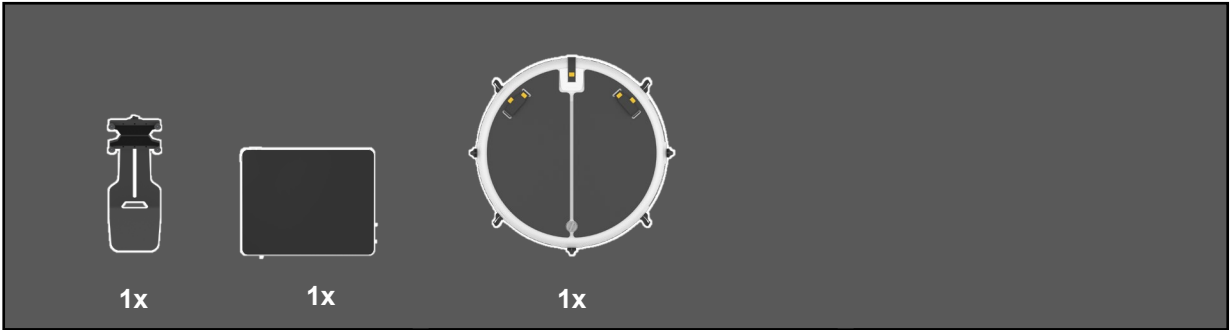


## 2.6

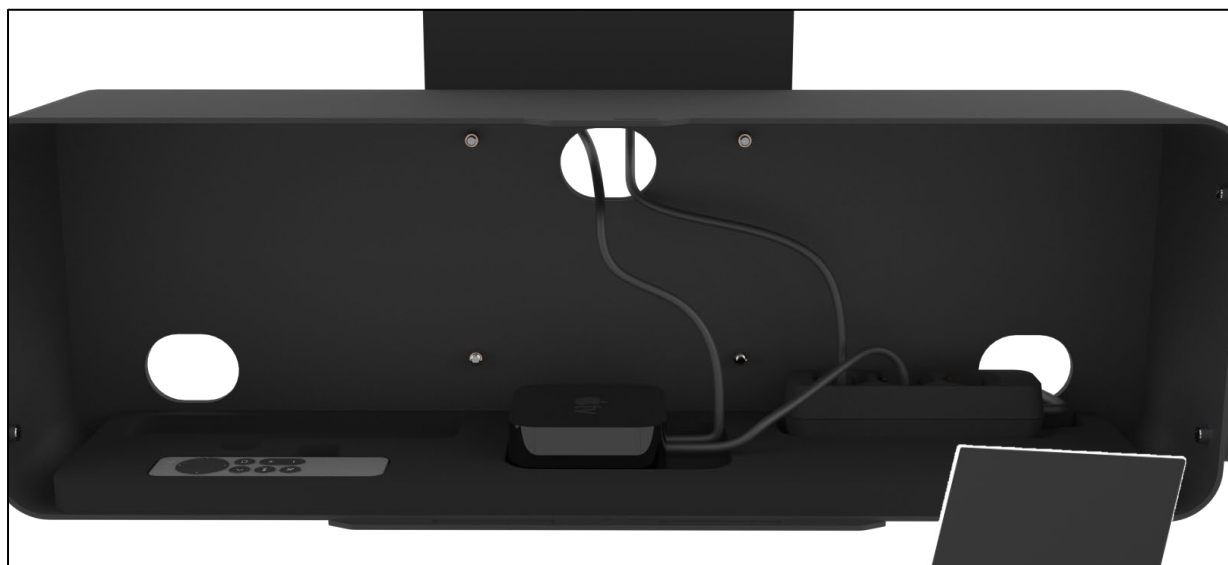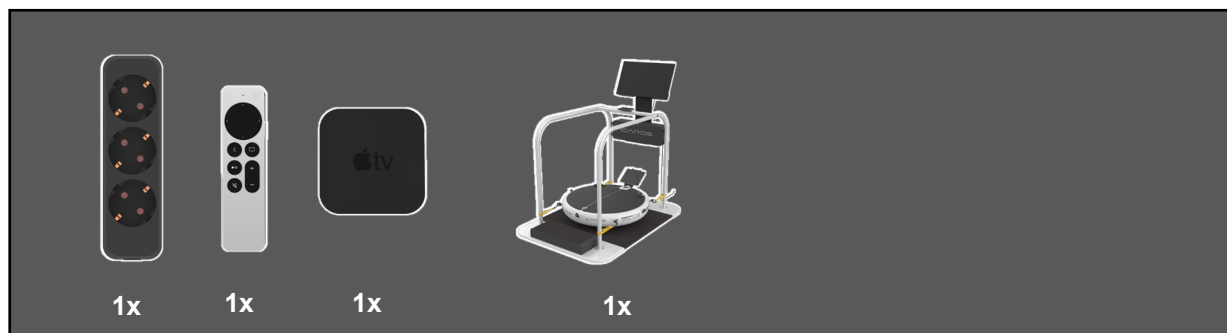

Place the Apple TV device, the multiple socket outlet and the remote controls of the Apple TV and the TV in the corresponding recesses.

Finally, connect the digital devices to each other and to the power supply.

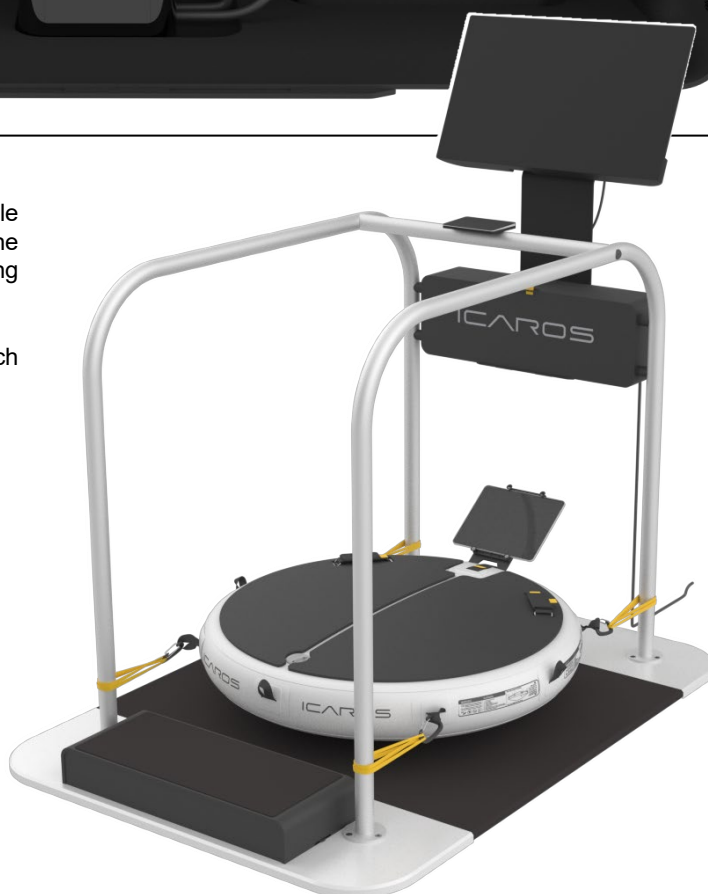

# Thank you

We are grateful that you have decided to purchase the ICAROS Guardian. The entire ICAROS team wishes you much success and pleasure in using it. If you require any further information about our products, please feel free to visit our website, contact us by phone or send us an email. We look forward to helping you.

**ICAROS GmbH**  
Fraunhoferstraße 5  
82152 Martinsried  
GERMANY

TEL +49 89 414 1821 00  
MAIL [info@icaros.com](mailto:info@icaros.com)  
WEB [www.icaros.com](http://www.icaros.com)